
Single Channel Film Dosimetry FilmQA XR

Andre Micke. Ph.D. amicke@ispcorp.com

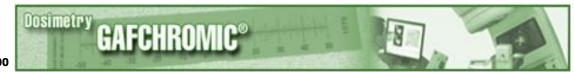




Outline

- Single Channel Dosimetry
- Best Operating Practice & Error Sources
- FilmQA XR Example




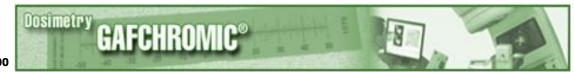


Single Channel Film Dosimetry

- Calibration Curve X=R
 $R_{ave} = R_{ave}(D) \leftrightarrow D = D(R_{ave})$
 - X_{ave} average film response
- Large calibration patches
 - Size min 2"x2" better 4"x4"



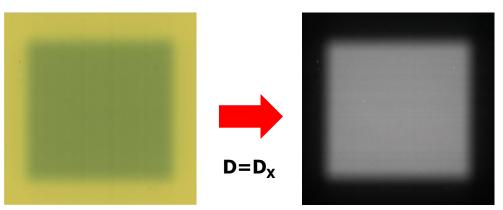

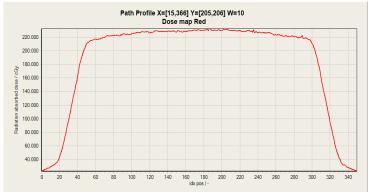



Scanner – Film Adjustment

Marguee


11"

10"

Hinge of the top cover

Scanner bed

Scanner lateral non-linear

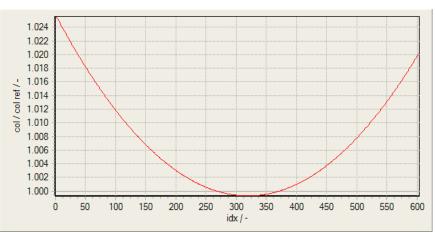

8.5" 8.5"x11" Full sheet of backgrouhd film or experimental film 8"x10" Portrait mode

- GAFChromic Film polarizes
- Light path includes mirrors
- Adjust film to Center in scan direction
- Use same film orientation

Andre Micke, October 2010






INTERNATIONAL SPECIALTY PRODUCTS 1361 Alps Road Wayne NJ 07470 · 973-628-4000



Scanning direction

Scanning Distortions

- Newton rings
- Cause: Scanner platen - film contact + moisture
- Signal variations 'amplified' by slope of calibration function
- Keep scanner platen clean and dry
- Hint of baking powder avoids scanner platenfilm contact

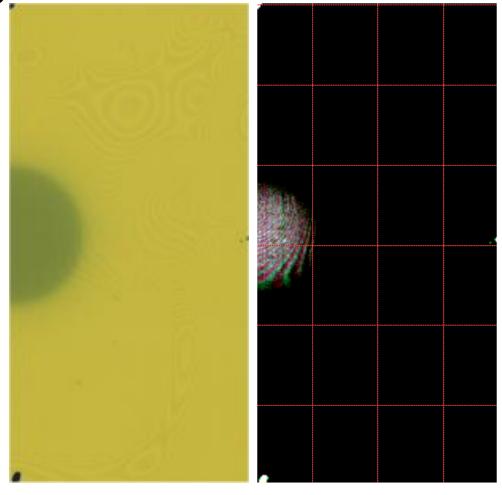




Dose Map





FilmQA XR - Evaluation Mode

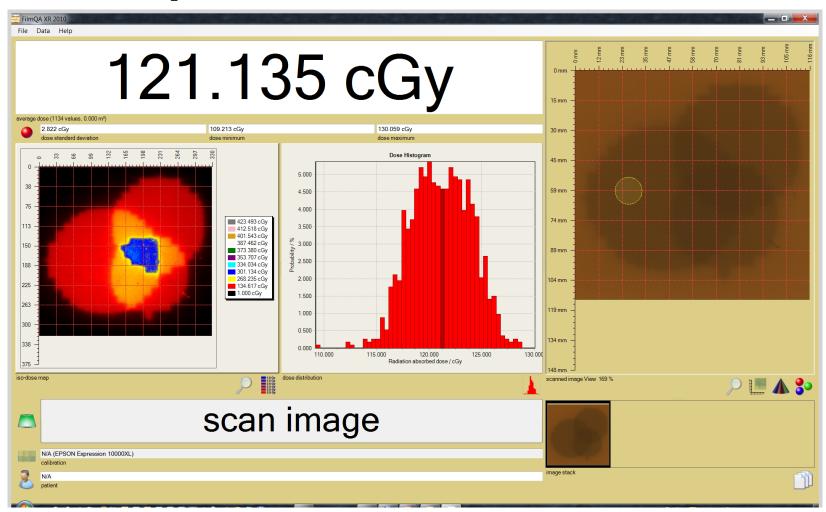


Iso-Map, Dose distribution histogram and Scanned XRQA film





FilmQA XR - Calibration Mode

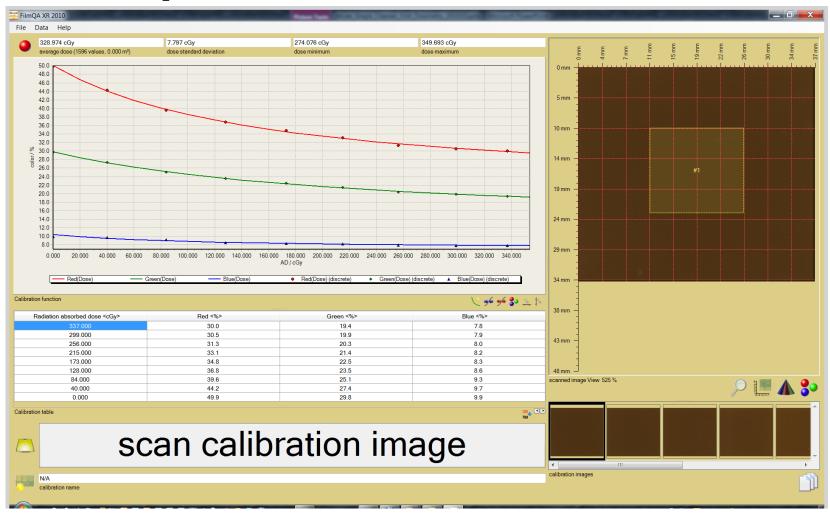

Calibration chart, calibration table and Scanned XRQA calibration patches



